ENA ETCH

9.6% Hydrofluoridric Acid

Indications:

9,6% Hydrofluoridric Acid, thixotropic for etching of porcelain veneers and inlays in porcelain before cementing with composite and for porcelain fractured surfaces to be restored with composite in dental laboratory.

3 ml syringe, 3 tips for application.

User:

Dentist and Dental technician.

Instructions for use:

1. Protective eyewear must be worn by users. Protective gloves must be worn by practitioner and assistant.
2. Remove syringe cap from the etching gel and place a pre-bent dispenser needle on the syringe. Carefully start the flow of the gel on a pad and immediately dispose of this sample. Do not use in patient's mouth. For maximum control of flow, do not push on plunger with thumb. It is recommended instead to hold syringe barrel between thumb and forefinger while resting plunger in palm of hand. Gently pull back with thumb and forefinger to start flow.
3. Carefully apply the gel to porcelain surface and leave for one minute while evacuating fumes with vacuum. Some ceramic materials are resistant to etching. To archive a surface suitable for bonding to those materials, air abrasion is recommended. Creating mechanical retention is advantageous. Cap syringe immediately after use.
4. Rinse treated surface with copious amounts of water for one minute and dry with clean, uncontaminated, oil-free and moisture-free compressed air.
5. Apply the silane (Porcelain primer) to treated surface. To promote rapid drying, use a gentle flow of clean, oil-free air, if desired. Cap the silane immediately after use. To check that the air is free of contamination, blow on the surface of a mirror: if any drops of water or oil appear, do not use it – Follow instruction for use for silane.
6. Proceed with the composite restoration or proceed with the luting of the treated ceramic appliance with composite cement.

Cautions:

The etching gel contains 9.6% hydrofluoric acid, a powerful and aggressive chemical that produces irritating fumes and must be handled with extreme care. Ingestion, contact with skin and eyes, and chronic inhalation of fumes are dangerous. Care must be taken to protect the user (with gloves and eyewear). This product has been buffered, gelled and packaged to minimize the dangers of hydrofluoric acid, provided it is used in accordance with these instructions.

Warnings: If the etching gel comes in contact with skin or soft tissue, rinse exposed area with copious amounts of water. Use high-speed evacuation system to remove water for 15+ minutes. Only for professional use. Storage at a temperature between 5°C and 27°C. Use at room temperature.

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MICERIUM S.p.A.