Venus® Diamond

Gebruiksaanwijzing	ℹ
Instructions for use	ℹ
Mode d'emploi	ℹ
Instrucciones de uso	ℹ
Istruzioni per l'uso	ℹ
Instruções de uso	ℹ
Gebruiksaanwijnzig	ℹ
Bruksanvisning	ℹ
Brugervejledning	ℹ
Bruksanvisning	ℹ
Käyttöohjeet	ℹ

Venus® Diamond

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Venus® Diamond

Instructions for use	ℹ
Medical Device – for use by dental health care professionals only.	
Intended purpose	
Venus Diamond is a light curing, radio-opaque nano composite. Therapeutic indications include adhesive, tooth-colored restorations for the anterior and posterior, including the occlusal surface. Due to the excellent material properties, a beautiful aesthetic result and a durable, high lustre polish is easily reached. Its radiopacity is equivalent to 2.5 – 3 mm of aluminum (dentine = 1 mm, enamel = 2 mm).	
Indications	
• Direct restorations of all cavity classes • Direct composite veneers • Correction of contacts of teeth (i.e. diastemas, combination defects in teeth, etc.) • Sealing of primary teeth • Core build-up • Repair of porcelain, composite (in combination with an adequate repair system) • Composite attachments for aligner treatment • Fixation of retainers	
Contraindications	
Application to dentine close to the pulp (less than 1 mm). In case of known or suspected allergy to components of the product, its use is contraindicated. Do not use this product in case of known or suspected allergies against (meth)acrylate compounds.	
Intended user	
For use by dental health care professionals only.	
Clinical benefit	
Kulzer medical devices ensure the rehabilitation of oral functions such as chewing, speaking and aesthetics. They stabilize the remaining dentition and/or the alveolar ridge restoratively or prosthetically.	
Potential side effects	
Hypersensitivities to the product or to its components cannot be excluded in individual cases.	
Composition	
Barium-Aluminum-Bor-Fluor-Silicate Glass, TCO-Urthencarylates, Silica, UDMA, TEGDMA, Titanium Dioxide, Fluorescent Pigments, Metallic Oxide Pigments, Organic Pigments, Ammoniumazoxiresaurer, BHT, Camphorquinone. Venus Diamond contains approximately 64% fill by volume. Its filler particle size is 5 µm – 20 µm.	

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