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PRECISION

NANOCORE DUAL

Dual-cured composite for post cementation and core build-up

INSTRUCTIONS FOR USE

General

NANOCORE DUAL is a dual resin composite, fluor realising, designed for post cementation and core build-up. NANOCORE DUAL may be used with any Bis-GMA compatible bonding agent.

Composition

Bis-GMA, 2-Hydroxyethylmethacrylate, riethylenglycoldimetacrylate, Benzol Peroxide, Co-Initiator, Photoinitiators, Dendritic Polymer, Barium aluminoborosilicate glass, Fluoride releasing filler, Fumed Silica, Nano Filler, White Pigment, Yellow Pigment shade A3, Alkyl Quaternary Ammonium Bentonite.

Cautions

NANOCORE DUAL contains polymerizable monomers which may cause allergic contact dermatitis in susceptible person. Wash thoroughly with soap and water after contact. If skin sensitization occurs, or if known allergy to methacrylate resin exists, discontinue use.

Precautions

Do not use in presence of eugenol based material; complete setting of NANOCORE DUAL may not take place. A calcium hydroxide or glass ionomer liner may be used to isolate NANOCORE DUAL from eugenol sources.

If refrigerated, allow to return to room temperature before use. Refrigeration will increase viscosity, making syringing more difficult, and will retard setting time.

Technical Data

Working time	1 minute and 30 seconds
Setting time	4 minutes and 30 seconds
Compressive strength	250 MPa
Diametral tensile strength	40 MPa
Flexural strength	68 MPa
E-Modulus	8 GPa
Water sorption	30 µg/mm ³
Solubility	2 µg/mm ³

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- After the endodontic canal treatment and the cleansing of the root canal (with the use of a sodium hypochlorite solution and with E.D.T.A.) and after the sealing of the third part of the apex, rectify the walls of the canal with the correct post in order to reach a layer of dentin reach in collagen fibres. Choose the correct fibre post for cementation.
- Proceed to etch the canal for approx 60 seconds with orthophosphoric acid at 32%- 40% (as Axia[®] Etch Precision). Clean with care, possibly with a needle in the cavity and dry well with paper points.
- As alternative to etch with orthophosphoric acid we suggest the use of Self Etching Agent Precision (in combination with the adhesive One-Q-Bond C.G.T. Precision): allow the product to sit on the dentin surface for at least 15 seconds, then remove the solvent with a blast of air for 10 seconds. You can also absorb the solvents utilizing paper points.
- Apply an adhesive, possibly dual or auto-polimerising, following the indication of the manufacturer. For best results mix equal parts of adhesive One-Q-Bond C.G.T. and Activator Precision (that makes the adhesive auto-cure) and apply a layer on the canal dentin. Allow the mix to sit for 15 seconds and then gently remove the solvents with a gentle blast of oil-free air or use paper points.
- Before mounting the new mixing tip and the intra-oral tip on the cartridge, extrude and eliminate a little quantity of composite before the use.
- Syringe NANOCORE DUAL into the canal, insert the post and light-cure for 20 seconds to stabilize the post.
- Continue to syringe NANOCORE DUAL into the prepared tooth, around the post, and complete build-up. Let material rise around the tip while withdrawing to eliminate air bubbles.
- Do not disturb NANOCORE DUAL after syringe placement or during the gelation stage. Light-cure for at least 40 seconds. Check for complete hardness. Chemical-cure intra-oral set-time is approximately 3 minutes from start of application. Set-time will be longer outside of the mouth.

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- Prepare the tooth according to clinical requirements.
- Isolate the tooth from oral fluids. A rubber dam is recommended.
- Acid etch the entire preparation with a 32%-40% o-phosphoric acid (Axia[®] Etch Precision) for 15 seconds.
- Rinse off etchant with a water spray for at least 15 seconds. Etched enamel should appear frosted.
- Apply a layer of adhesive on the prepared tooth surface. Suggestion: mix equal part of One-Q-Bond C.G.T. Precision with Activator Precision, that makes the adhesive auto-cure. When using another adhesive, follow instructions for use with dual-cure composites.
- Allow mixture to sit for 15 seconds and then remove residual solvents with a gentle blast of oil-free air. Repeat the application of One-Q-Bond C.G.T. with Activator at least once on the prepared tooth surface. It is not necessary to wait 15 seconds between each additional application,

however, solvent evaporation is necessary between application. Light-cure for 20 seconds after the final application of adhesive agent.

- Before inserting the mixing tip and the intra-oral tip on the cartridge, extrude and eliminate a little quantity of composite before the use.
- Place intra-oral tip of assembled cartridge into the base of prepared tooth and dispense an adequate amount of material to complete build-up. Let material rise around the tip while withdrawing to eliminate air bubbles.
- Do not disturb NANOCORE DUAL after syringe placement or during the gelation stage. Light-cure for at least 40 seconds. Check for complete hardness. Chemical-cure intra-oral set-time is approximately 3 minutes from start of application. Set-time will be longer outside of mouth.

For the version in Dispenser: on the rare occasion that the orifice of the intra-oral tip gets clogged, cut at the ending the last millimetres off with a pair of scissors and continue to express the material.

Storage

Store the product < 24°C. Do not use after expiration date. Refrigerate for longer shelf life.

Packaging

- NanoCore Dual in Dispenser
1 cartridge of 50 ml, 25 mixing tips, 25 intra-oral tips.
Shades: A3, White.

- NanoCore Dual in Syringe Automix
2 syringes of 5 ml cad, 15 mixing tips, 10 standard intra-oral tips,
10 canalar intra-oral tips. Shades: A3, White.

For dental use only. Keep out of reach of children.

PRECISION products are carefully developed and extensively tested. We therefore guarantee their impeccable quality when used in accordance with their instructions for use. Application, use and processing of the product are beyond our control and are subject to the responsibility of the user.

For more information please contact:
DENTALICA S.p.A. – Via Rimini 22 – I-20142 Milano – Italia
Tel.#39 - 02895981 – Fax #39 -0289504249
www.dentalica.com – E-mail: dentalica@dentalica.com

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Composition

Bis-GMA, 2-Hydroxyethylmethacrylate, riethylenglycoldimethacrylate, Benzol Peroxide, Co-Initiator, Photoinitiators, Dendritic Polymer, Barium aluminoborosilicate glass, Fluoride releasing filler, Fumed Silica, Nano Filler, White Pigment, Yellow Pigment shade A3, Alkyl Quaternary Ammonium Bentonite.

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