Quick post hygiene treatment

GB **BlancOne® CLICK** is a quick post dental hygiene whitening treatment. This treatment offers a very popular upgrade to the dental hygiene session at a very low cost and in just 10 minutes. **BlancOne® CLICK** has been designed to promote a complete whitening treatment (Touch or Ultra) and as a promotional and marketing tool for the office.

USE INSTRUCTIONS

A) Preparing the gel

1. Remove the caps of the two syringes (whitening gel and activating gel) and connect them together using the luer-lock connector.
2. Gently push the piston of one syringe to completely extrude the gel into the other syringe.
3. Now push the piston of the syringe with the gel, transferring it into the empty syringe.
4. Repeat the operation described in the previous step until the two gels (whitening and activator) are completely mixed, creating a single homogeneous gel.
5. Remove the connector from the syringe with the gel and attach the applicator tip.

B) Preparing the patient

1. The patient should be fitted with a BlancOne® gag with tongue depressor and then insert cylindrical tampons above and below the lips to isolate them from the area to be treated.
2. Remove the salivary film with a brush; if necessary, use a mix of water and pumice stone. DO NOT use polishing pastes or products containing fluorine. Rinse and dry.
3. Measure the patient's initial shade of white using the shade guide.

C) Whitening treatment

1. Gently push the piston of the syringe containing the whitening gel to apply the gingival gel on the surface of all teeth to be bleached.
2. Cure the whitening gel with a blue light. Two types of lights can be used:

• Curing light

- i. Use a powerful light (min. 1500 mW/cm²) to irradiate each tooth for 20". It is possible to activate 2 teeth at a time on the lower central arch.
- ii. After completing both arches, repeat the previous procedure 2 or 3 times until the gel has been completely activated (see figure).

• Arc light

Use the BlancOne® ARCUS arc light to activate both arches simultaneously. Position the head of the light close to the teeth and activate for 10 minutes until the gel has been completely cured (see below).

When used according to BlancOne protocols, the ARCUS whitening lamp is safe for teeth and gums.

Protect lips with UV-protection factor balm.

The lamp generates heat on the skin similar to a tanning lamp. If the patient feels a little bother, move the lamp slightly away from the mouth.

The gel will be properly cured depending on:

- Lamp power (mW/cm²) – The higher the power the faster the curing operation.
- Gel thickness – Apply a uniform and thin layer of gel (1-2 mm); thicker layers will slow down the curing process.
- Distance from the light source – The distance between the light and the gel must be as short as possible; increasing the distance will increase the curing time.

Once activated, the gel will be swollen and full of oxygen (bubbles) as shown in the figure.

3. Remove the excess whitening gel using a cylindrical cotton roll and suction away the gel residues, rinsing with the air/water syringe.

WARNING: The activated gel will be rather dense. If the suction spout becomes clogged, suction off with copious amounts of water.

NOTE: the gingival edge might be slightly red in some points but this is a temporary effect that will disappear in about one hour.

D) End of treatment

1. Use the shade guide to measure the final shade of white of the patient's teeth.
2. Remove the gag and rinse the patient's mouth.

WARNINGS

- 6% of H₂O₂ is released.
- Not to be used on anyone under 18 years of age.
- To be sold only to dental practitioners.
- To be used only by dental practitioners or under their direct supervision if an equivalent level of safety is ensured.
- Keep teeth bleaching products away from children.
- Avoid contact with skin, mucous membranes and eyes. In case of contact with EYES: rinse eyes carefully with water for several minutes and request medical advice/attention. ON SKIN/ MUCOUS MEMBRANES: wash with copious amounts of water. If irritation occurs request medical advice/attention.
- If swallowed in small quantities the product is harmless, otherwise consult a doctor.
- Some users may experience sensitivity or gingival irritation. While these are mild and transient in most cases, sometimes treatment by a dentist may be required.

PRECAUTIONS

- A dental examination is mandatory prior to bleaching.
- Bleaching will not lighten porcelain crowns and dental restorations.
- For at least 6 hours after the bleaching procedure do not use tobacco products and do not drink coffee or red wine.
- Not intended for use by pregnant women. Consult a doctor if you are breast feeding.
- Do not swallow the gel. Avoid contact with tissues other than teeth.
- Acidic foods/juices may cause sensitivity if consumed shortly after bleaching.
- Gum tissue irritation may result if used in individuals with periodontal disease.

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