

Prime&Bond active™

UNIVERSAL ADHESIVE



Caution, consult
instructions for use

20 sec Agitate slightly	5 sec Remove solvent	Cure ≥ 500 mW/cm²: 20 sec ≥ 800 mW/cm²: 10 sec



Manufactured by:
DENTSPLY DeTrey GmbH,
78467 Konstanz, Germany

© DENTSPLY DeTrey
2016-01-11
www.dentsply.eu
www.dentsply.com



DENTSPLY

Step by step instructions:

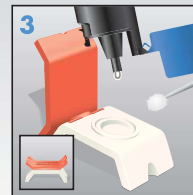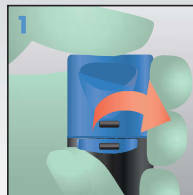

Preparation

Pull off originality seal.

Flip Top Bottle can be opened and closed with one hand. Do not screw off the cap.


Dispense **Prime&Bond active™** into a CliXdish™. In a closed CliXdish™ **Prime&Bond active™** remains usable for up to 30 minutes.

To close the bottle, press down the cap until it locks in place with a perceptible "click".

Cleaning & Disinfection

Disassemble CliXdish™ for reprocessing.

OR

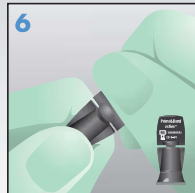

Single Unit Dose

Grasp container at each end, placing thumb along center score. Firmly apply pressure until container separates.



Insert applicator into opening to saturate applicator tip.

OPTIONAL



Etching

Start etching at enamel margins.







Extend to dentin.



Rinsing

Remove acid gel and rinse.

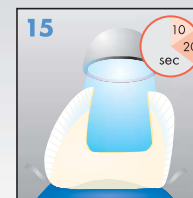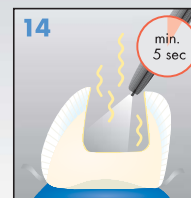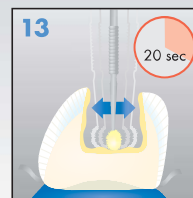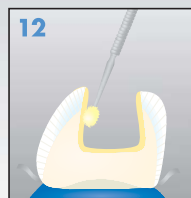

Application

Remove excess water.
Do not desiccate!

Apply **Prime&Bond active™** to all cavity surfaces.
Avoid pooling.

Keep **Prime&Bond active™** slightly agitated for 20 seconds.

Evaporation

Evaporate solvent with air for at least 5 seconds.

Light-Curing

$\geq 500 \text{ mW/cm}^2 = 20\text{s}$
 $\geq 800 \text{ mW/cm}^2 = 10\text{s}$